

medilines

Faecal Fat Colour

REAGENTS medilines Faecal Fat Colour

METHOD

- Prepare a stool smear on the glass slide
- Wet the smear with 3-4 drops of **medilines** Faecal Fat Colour.
- Cover it with a cover slip.
- Warm briefly over a low flame.
- Examine under the microscope.

RESULT

Neutral Fat	Red
Fatty Acid Plaques	Red

NORMAL On a mixed diet, every stool contains a few fat plaques and soap crystals.

REFERENCES

- E. Grabener, Praxislaboratorium, 5th ed., Georg Thieme Verlage, Stuttgart 1966, p.109*
L. Hallmann, Klinische Chemie und Mikroskopie, 10th ed., Georg Thieme Verlage, Stuttgart 1966, p.56
N. Henning, Klinische laboratoriumsdiagnostik, Verlag Urban & Schwarzenberg, Berlin-Munchen-Wien 1966, p. 525
B. Romeis, Mikroskopische Technik, 16th ed., R. Oldenbourg Verlag, Munchen-Wien 1968, p. 254, 261